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AR 1 Series

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AR 1 Series

Table of Contents

Importer	2
Product Summary	3
Product Specification	4
Product Detail	5
Use and Control	6
Safety Precautions	8
Warranty	9

Thank you for choosing COEPIC.

Please be informed of the instructions before use.

Please read the “Safety Precautions”

Importer

No.	Category	Description
1	Company Name	AURORA S Co., Ltd.
2	Address	#401 Cresyn Tower III, 375, Achasan-ro, Gwangjin-gu, Seoul
3	Customer Service	02-455-7117

Product Summary

No.	Category	Description
1	Item Type	Machine or Instrument for Generating Electrical Stimulation, Medical Use
2	Item Name	Low Level Light Therapy Device
3	Product Name	COEPIC (코에픽)
4	Model Name	AR 1
5	Category Name (Class)	A16060.02 [Class II]
6	License No.	Product License No. 14-3457
7	License Date	2014-12-03
8	Regal Manufacturer	ANDON HEALTH CO., LTD (No. 3 Jinping Street, Ya An Road, Nankai District, Tianjin, China)
9	Factory	ANDON MEDICAL CO., LTD. (No. 26 Hangyu Road, Tianjin Airport Economic Area, Tianjin 300380, China)
10	Packaging Unit	1 Set
11	Intended for use	The AR 1 is an innovative and effective treatment for the relief of those irritating symptoms associated with allergic rhinitis like watery eyes, runny nose and sneezing.


Product Specification

Category	Description	Remarks
Internal Power	DC 3V (1.5V X 2)	
Treatment Method	Dual wavelength phototherapy	
Wavelength (Frequency)	652 nm (nanometers) & 940 nm (nanometers)	
Output Level (Light Intensity)	3~6mW for each LED (for each probe inserted into the nose)	
Diameter of Radiation	7 mm \pm 5%	
Output Stability	\pm 15%	
Output (Lighting) Mode	Continuous mode	
Operating Time	Approx. 3 minutes	
Alarm	It rings once at the beginning and end of operation / It rings twice when treatment is finished.	
Battery Type	2 x 1.5V/ AAA batteries	
Battery Life	Approx. 100 sessions (3 minutes per session)	
Electric rating / Type and level of protection from an electric shock	Internally powered device, BF-type device	
Storage Conditions	Temperature -20 ~ 60[°C], Humidity 30 ~ 85[%], Atmospheric Pressure : 70kPa ~ 105kPa	
Operational Conditions	Temperature 15 ~ 40[°C], Humidity 30 ~ 85[%], Atmospheric Pressure : 80kPa ~ 105kPa	

Product Detail

A. Main body

Order	Part
①	Main Body
②	On/Off Button
③	Probe
④	Connector (Socket)
⑤	Connector (Plug)
⑥	Battery Cover
⑦	Clamp
⑧	Probe Cover
⑨	Inside-bush
⑩	LED Chips

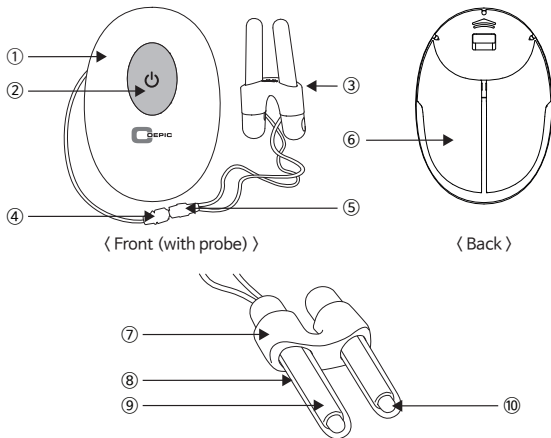〈 Front (with probe) 〉

〈 Back 〉

〈 Probe 〉

Use and Control

A. Before Use

- ① Read the manual before use and study the instructions and safety precautions.
- ② Check if the power switch is set to OFF.
- ③ Check if the batteries have been inserted correctly.
- ④ Remove any liquids around the device to avoid spilling and damaging the device.
- ⑤ Check if both connectors (socket, plug) have been connected properly.
- ⑥ Check if there are flammable gases nearby before using the device as electric sparks may lead to ignition of flammable liquids and gases.

B. Use and Control



1) Inserting the battery

- ① Open the battery cover pushing in the direction of the arrow on the back of the product.

- ② Insert two AAA batteries according to the polarity symbols (+ and -).

- ③ Close the battery case once the batteries have been inserted.

* Flickering ON/OFF button means the battery is low and batteries must be replaced.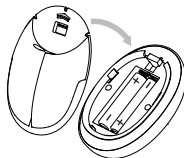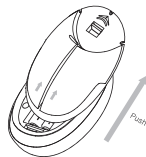

Use and Control

2) How to use the device

- ① Hold the main body with one hand and separate the probe from the main body with the other hand.
- ② Insert the probe inside the nose. Forceful insertion of the probe may be dangerous.
- ③ Turn on the device by pressing the ON/OFF button.
- ④ The two wavelengths are activated and the probe produces LED rays.
- ⑤ One treatment session is approximately 3 minutes long.
- ⑥ The device will turn off automatically after the session.
 - * To pause the device during treatment, press the ON/OFF button.
 - * It is recommended to use the device 3 to 5 times a day, 3 minutes per session, for at least 14 days.
 - * When experiencing severe symptoms of rhinitis, use more frequently than the number of sessions recommended for use.

C. Storage and Maintenance

- ① Store at room temperature and avoid hot, humid places.
- ② Avoid places that will have a negative impact on the product due to atmospheric pressure, temperature, humidity, and wind.
- ③ Clean the surface and the probe with a cloth, and do not use cleaning products.
- ④ Keep safe from vibration and shock.
- ⑤ Do not store with chemicals and gases.
- ⑥ Remove battery when product will not be in use for a long period of time.
- ⑦ Always leave repairing to professionals. Do not repair the product yourself.

Safety Precautions

A. Warning

: Electric sparks may cause ignition of flammable liquids and gases due to ignition electrodes.
Check to make sure the device is away from flammable gases before use.

B. Age

: Children under the age of 16 should operate the device only under the supervision of a guardian (or adult), sitting or standing straight while fully awake as the guardian (or adult) watches the entire procedure.

C. General Warning

- ① Do not look straight into the laser from the probe.
- ② Do not share the probe to prevent cross-contamination.
 - * User must not share the product with family members or other individuals to prevent possible infection.
- ③ Leave repairing to professionals in case of product breakdown.

D. Usage

- ① Use only with a doctor's prescription and order. Individuals with the following conditions must consult a doctor before use.
 - A person with liver and/or cardiac diseases
 - A pregnant woman or nursing woman
- ② Do not use if under the influence of alcohol and/or when with reduced physical and mental capabilities.

E. Caution

- ① Do not forcefully insert the probe into the nose.
- ② Be careful not to block the nasal cavity (nose).
- ③ Do not lay down or sleep while using the device.

F. Prevention of Safety Accidents

- ① Keep device away from children.
- ② Do not use the device with wet hands.
- ③ Do not rebuild device.
- ④ Remove battery when product will not be in use for a long period of time.

Warranty

Certificate of Warranty

1. This product is distributed after a strict quality check and inspection by the company.
2. The warranty period is 1 year from the purchase.
3. This certificate of warranty cannot be reissued.
4. This warranty does not cover any damage caused by the user.

Model Type	AR 1
Manufacturer's Serial No.	
Date of Purchase	
Customer Name	
Address	
Contact	
Salesperson	

For any product-related inquiries, please contact the seller or the customer service team.

#401 Cresyn Tower III, 375, Ahasan-ro, Gwangjin-gu, Seoul

TEL : 02-455-7117

www.coepic.co.kr

Date (MM/DD/YY) : 02/10/2020